

ONE TEST, TWO RESULTS
**GIARDIA/CRYPTOSPORIDIUM
 QUIK CHEK™**

Simple and accurate performance

- Simultaneous detection and differentiation of *Giardia* and *Cryptosporidium* using one test
- Less labor intensive than traditional methods
- Non-subjective results in 30 minutes without instrumentation



GIARDIA/CRYPTOSPORIDIUM QUIK CHEK™

Product Order Information


Product Name	Cat. No.	Sample Type	Time to Result	Format
GIARDIA/CRYPTOSPORIDIUM QUIK CHEK™	T30407	Fresh and frozen fecal sample (unpreserved), 10% formalin, SAF, Cary Blair, C&S	< 30 minutes	Rapid EIA

GIARDIA/CRYPTOSPORIDIUM QUIK CHEK™


Assay Protocol

- 1 Dilute fecal specimen**
For fresh or frozen specimens, add 25 μ L of specimen to tube containing 500 μ L of *Diluent* and 1 drop of *Conjugate*. Mix thoroughly.


For preserved specimens, add 100 μ L of specimen to tube containing 400 μ L *Diluent* and 1 drop *Conjugate*. Mix thoroughly.

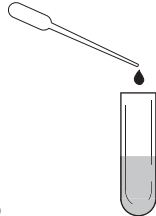


- 2 Add sample**
Transfer 500 μ L of diluted sample-conjugate mixture to *Sample Well*. Incubate at room temperature for 15 minutes.

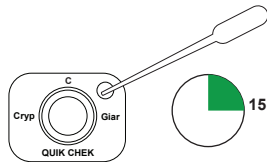

- 3 Wash Membrane Device**
Add 300 μ L *Wash Buffer* to *Reaction Window*. Allow to completely absorb.

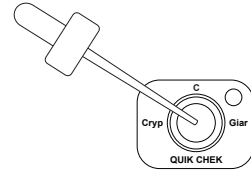


- 4 Add Substrate reagent**
Add 2 drops *Substrate* to *Reaction Window*. Incubate at room temperature. Read at 10 minutes after addition of *Substrate**.

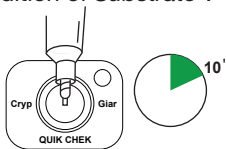


*A positive result may be interpreted at any time during the 10-minute incubation. A test cannot be ruled as negative or invalid until after the 10-minute incubation.


Easy to Read Results

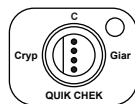


Positive Crypto. Result

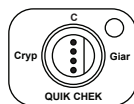

Positive Giardia Result

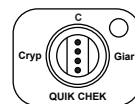


Positive Crypto. and Positive Giardia Result

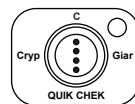

Negative Result

If there are no dots at C (control) the test is invalid. See package insert for additional interpretation of results.

GIARDIA/CRYPTOSPORIDIUM QUIK CHEK™ Package Insert

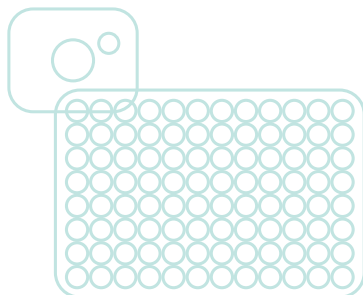

Developed and manufactured by

To contact a **TECHLAB®** Representative

Call +1-540-953-1664 or visit www.techlab.com



PROVEN DIAGNOSTIC PERFORMANCE

TECHLAB, Inc. is a leading developer and manufacturer of infectious disease diagnostics products worldwide. Since 1989, the company has offered a quality portfolio of diagnostic tests which now includes assays for *C. difficile*, *Campylobacter*, Shiga toxin producing *E. coli*, *H. pylori*, protozoan parasites, and biomarkers of intestinal inflammation.

