

COMPLETE RESULTS

C. DIFF QUIK CHEK COMPLETE®

Simultaneously detects GDH and Toxins A and B

- Detects *C. difficile* glutamate dehydrogenase antigen and toxins A and B in a single reaction well
- Yields results in 30 minutes
- Indicates whether *C. difficile* bacteria are present **and** producing toxin
- Provides a clean background and strong signal with patented upflow technology



C. DIFF QUIK CHEK COMPLETE®

The C. DIFF QUIK CHEK COMPLETE® test


Product Order Information

Product Name	Cat. No.	Sample Type	Time to Result	Format
C. DIFF QUIK CHEK COMPLETE®	T30525C	Fresh, frozen, transport media (Cary Blair or C&S)	<30 minutes	Rapid EIA


C. DIFF QUIK CHEK COMPLETE®

Assay Protocol


- 1** Add to a test tube:
- 750 μ L *Diluent*
 - 1 drop *Conjugate*
 - 25 μ L (1st graduation) of specimen
 - Mix thoroughly

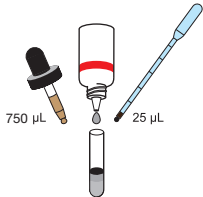


- 2** Transfer 500 μ L (highest graduation) from tube to small *Sample Well*. Keep the cassette at room temperature and wait 15 minutes.

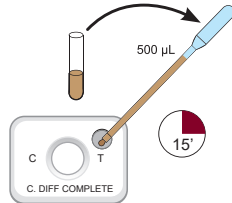

- 3** Add 300 μ L *Wash Buffer* to large *Reaction Window*. Allow to completely absorb.

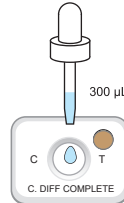


- 4** Add 2 drops *Substrate* to large *Reaction Window*. Keep the cassette at room temperature and wait 10 minutes. Read results.

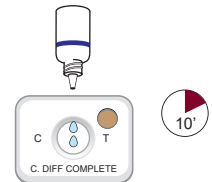


Easy-to-read Results


- Results in 30 minutes without instrumentation
- Uses the popular TECHLAB® QUIK CHEK™ format
- Simple and clear interpretation of results
- Refer to the package insert for additional interpretation of results.

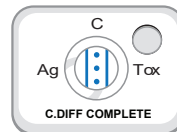

**Positive Ag
Positive Tox**
C. difficile is present
and producing toxins

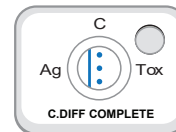


**Positive Ag
Negative Tox**
C. difficile is present

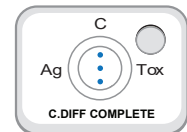

Negative
C. difficile is not
present

C. DIFF QUIK CHEK COMPLETE Package Insert.

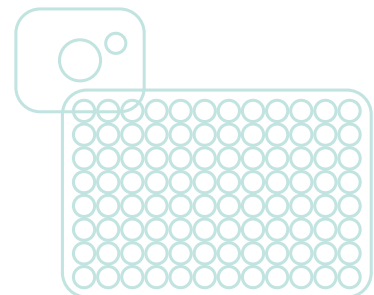

Developed and manufactured by



PROVEN DIAGNOSTIC PERFORMANCE

To contact a **TECHLAB®** Representative

Call +1-540-953-1664 or visit www.techlab.com

TECHLAB, Inc. is a leading developer and manufacturer of infectious disease diagnostics products worldwide. Since 1989, the company has offered a quality portfolio of diagnostic tests which now includes assays for *C. difficile*, *Campylobacter*, Shiga toxin producing *E. coli*, *H. pylori*, protozoan parasites, and biomarkers of intestinal inflammation.



All trademarks referenced are trademarks of TECHLAB, Inc. PN 7192021013