

● Colorex™ C.difficile



For detection of *Clostridium difficile*

Colorex™

Ready to use plates made with the original CHROMagar™ powder base



Plate Reading

- *C. difficile*
→ colourless and fluorescent under UV light at 365 nm
- Other bacteria
→ colourless, no fluorescent or inhibited

For detection of *Clostridium difficile*

Background

Clostridium difficile is the leading cause of nosocomial infectious diarrhea in adults. These infections occur mostly in patients who have both medical care and antibiotic treatment. The symptoms of *C. difficile* infection are fever, abdominal cramps and severe diarrhea leading to death. In the United States, nearly 250,000 people each year develop *C. difficile* infections with at least 14 000 deaths (CDC estimate, 2013). Due to the emergence of highly toxigenic *C. difficile* strains, these infections have become more frequent and more difficult to treat in the last years.

Although PCR has become the leading *C. difficile* detection technique, culture is essential for strain typing and antimicrobial susceptibility testing. Colorex™ *C.difficil* is a fluorogenic culture medium, extremely sensitive and selective, especially designed to simplify and speed up (24h) the culture of *C. difficile*.

Medium Performance

- 1 RAPID DETECTION COMPARED TO TRADITIONAL MEDIA**
Big colonies (around 2mm) of *C. difficile* after only 24 h of incubation in anaerobic atmosphere, contrary to traditional media requiring 48 h.
- 2 HIGH SENSITIVITY AND SPECIFICITY**
C. difficil is detected by characteristic fluorescent colonies (under UV light at 365 nm) and the specimen's flora largely inhibited
- 3 POLYVALENCE**
This medium can be used for clinical specimens as well as environmental samples.


Specificity / Sensitivity: ≈ 100 %*

* Sensitivity from scientific studies: Gaillot O. et al. 100 % (40/40), Van Broek et al. 100 % (95/95)

Medium Description

Powder Base	Total	54.7 g/L
	Agar	15.0
	Peptone & Yeast extract	25.0
	Salts	9.0
	Growth factors	4.0
	Chromogenic mix.....	1.7
	Storage at 15/30 °C - pH: 7.8 ± 0.2	
	Shelf Life	2 years
Supplement	Powder form.....	325 mg/L
	Storage at 2/8 °C	Shelf Life 2 years

Usual Samples	Stool, Environmental
Procedure	Direct streaking or after an appropriate enrichment step of the sample. Incubation at 37 °C for 24 h. Anaerobic condition.

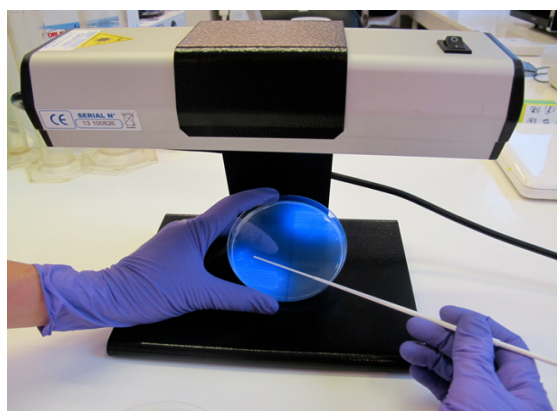

Distributed by

analytichem
your science enabled

AnalytiChem USA, Inc.
P.O. Box 788 Waterville
Maine, U.S.A, 04903-0788
Toll Free Phone: +1 (866) 591-7120
Email: info.us@analytichem.com

AnalytiChem Netherlands B.V.
Communicatieweg 7
3641 SG MIJDRECHT, The Netherlands
Telephone: +31 (297) 286848
E-mail: info.nl@analytichem.com

Ready to use plates made with
the original CHROMagar™ powder base

CHROMagar, Paris - France
www.CHROMagar.com